

# Installation Guide for STable (©) Basics II IVF Table

- 1) Position the leg system on its desired place.
- 2) On the top of the damping plate there is a built-in small spirit level. That is the front side of the damping plate where you can see this round spirit level.
- 3) Put the damping plate on its place in the leg system. On the bottom side of the damping plate there are four steel pins. There are holes in the leg system to receive the bottom side positioning pins of the damping plate. These four pins position the damping plate and the gravity holds it in the leg system.
- 4) Put a separate spirit level on the top plate, not on but next to the damping plate, either on the stainless steel covered or on the plexiglas covered area.
- 5) In the bottom of the four legs, close to the floor, there are height adjusting screws of the leg system. With a 13 mm wrench you can adjust the exact horizontal position of the leg system, by watching the spirit level.
- 6) The final step is the adjustment of the horizontality of the damping plate. For this purpose you will find four height adjustment nuts around the bottom positioning pins of the damping plate. You can adjust them by the 13 mm wrench, by watching the small built-in spirit level of the damping plate. You can tilt the damping plate by its height adjustment nuts in comparison to the leg system.